
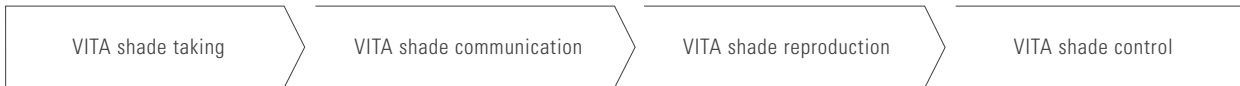


# VITA Bleachedguide 3D-MASTER®

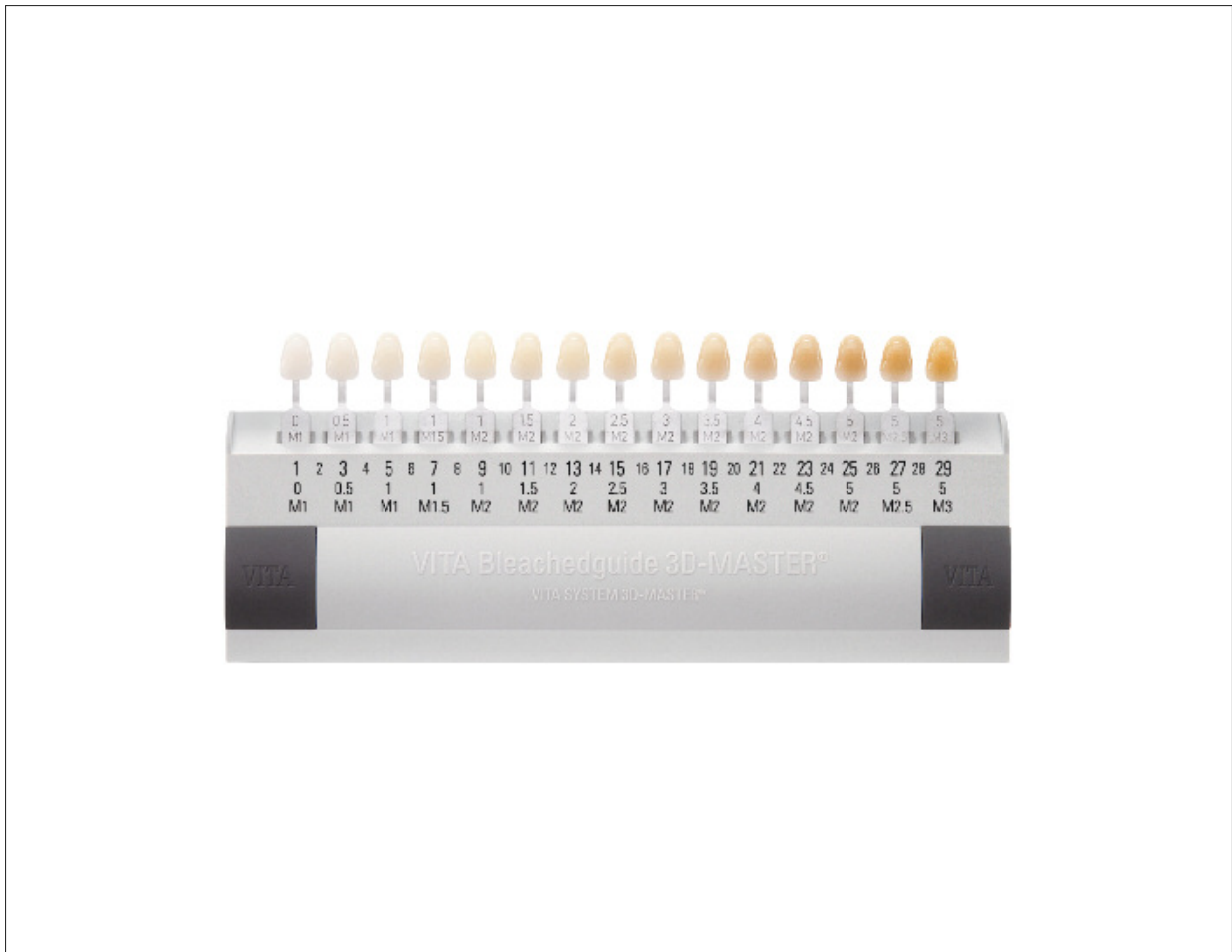

Date of issue: 03.13

VITA shade, VITA made.

**VITA**

## Product description


The VITA Bleachedguide 3D-MASTER is the best way to plan and monitor tooth whitening processes. It has a logical structure, similar to that of the VITA SYSTEM 3D-MASTER, based on the three parameters of brightness (value), chroma (saturation) and hue (color).

In addition, the 29 shade levels defined by the American Dental Association (ADA) are integrated in the VITA Bleachedguide 3D-MASTER. The VITA Bleachedguide 3D-MASTER uses this globally recognized standard for shade level assessment and is therefore recommended for every dental professional.

## Safe and effective cosmetic dentistry



Hold the VITA Bleachedguide 3D-MASTER at a distance of about 50 cm from the patient's open mouth.

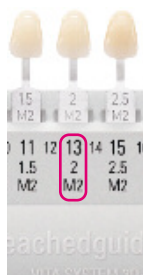

Determine the current brightness of the teeth and provide this either with the VITA SYSTEM 3D-MASTER or using the numerical codes.

Perform the teeth whitening procedure with the appropriate products.



Check the treatment results.

If the actual result differs from the desired value, the treatment process can be repeated.

### Advantages

- The clear organization – now with additional numerical designation – supports communication between dentist and patient. The numerical grades of the VITA 3D-MASTER Bleachedguide are based on the brightness levels defined by the American Dental Association, and facilitate accurate recording of grades in the bleaching process.
- The desired result of the tooth bleaching procedure is clearly defined with the patient prior to treatment using sample color sticks, and then checked after treatment has been completed.
- The appropriate color for the treatment is determined with the VITA Toothguide 3D-MASTER and achieved using the appropriate materials.

Description	Content	Art. no.
VITA Bleachedguide 3D-MASTER	1 piece	B361

Printed matter	Description	Order no.
VITA Bleachedguide 3D-MASTER	Instructions	920-26266
VITA Bleachedguide 3D-MASTER	Brief instructions	920-26270
VITA Bleachedguide 3D-MASTER	Product Sheet	1432E
VITA Bleachedguide 3D-MASTER	Paper sample	1486E

**VITA**