

VITA VMK Master®

For veneering metal substructures in the conventional CTE range



VITA shade taking

VITA shade communication

VITA shade reproduction

VITA shade control

Edition: 11.11

VITA shade, VITA made.

VITA

Product description

VITA VMK Master is a feldspar veneering ceramic for metal substructures in the conventional CTE range made of high gold content, reduced gold content and palladium-based alloys. The ceramic is particularly suitable for veneering non-precious metal substructures thanks to its firing temperature and chemical-physical properties.

Advantages

- Uncompromising material with regard to reliability and esthetics
- Improved "Master" manufacturing process (Material Stabilization for Efficient Reproduction) for unsurpassed processing characteristics such as enhanced modelling properties, minimized shrinkage behavior
- Classic and familiar dentine-incisal layering pattern
- Comprehensive range of additional materials for individualization
- Highly esthetic restorations thanks to brilliant shades
- Available in VITA SYSTEM 3D-MASTER and VITA classical A1–D4 shades

VITA VMK Master® Sets – VITA SYSTEM 3D-MASTER® shades



VITA VMK Master STANDARD SET 3D-MASTER

Standard set for the reproduction of the 3D-MASTER shades
Prod. No. BVMKSET3D**/**

Quantity	Content	Material
1	12 g	WASH OPAQUE WO
5	12 g	OPAQUE OP1-OP5
26	12 g	OPAQUE DENTINE 1M1-5M3
26	12 g	DENTINE 1M1-5M3
2	12 g	ENAMEL EN1, EN3
1	12 g	TRANSLUCENT T4
1	12 g	WINDOW WIN
3	12 g	CERVICAL CE1-CE3
5	12 g	NECK N1-N5
2	12 g	CORRECTIVE COR1, COR2
1	50 ml	VITA MODELLING FLUID
1	50 ml	VITA MODELLING FLUID RS
1	50 ml	VITA OPAQUE FLUID
1	–	Shade indicator
1	–	VITA Linearguide 3D-MASTER
–	–	Accessories
1	–	Working instructions

* also available as 10-COLOR SET (Prod. No. BVMK10S3D) with the following ten shades: (1M2, 2L1.5, 2M1, 2M2, 2R2.5, 3L1.5, 3M1, 3M2, 3M3, 4M2)

** each also available with PASTE OPAQUE (Prod. No. BVMKSETP3D / Prod. No. BVMK10SP3D)



VITA VMK Master STARTER SET 3D-MASTER (2M2, 3M2)

Two-color set

Prod. No. BVMKSS3D*

Quantity	Content	Material
2	12 g	OPAQUE OP2, OP3
2	12 g	OPAQUE DENTINE 2M2, 3M2
2	12 g	DENTINE 2M2, 3M2
1	12 g	ENAMEL EN1
1	12 g	TRANSLUCENT T4
1	50 ml	VITA MODELLING FLUID RS
1	50 ml	VITA OPAQUE FLUID
2	–	Shade tabs 2M2, 3M2
1	–	Working instructions

* also available with PASTE OPAQUE (Prod. No. BVMKSSP3D)



VITA VMK Master BLEACHED COLOR SET 3D-MASTER*

Ultra-bright colors for the reproduction of bleached teeth

Prod. No. BVMKBCS

Quantity	Content	Material
1	12 g	OPAQUE OP0
3	12 g	OPAQUE DENTINE 0M1-0M3
3	12 g	DENTINE 0M1-0M3
1	12 g	ENAMEL EN1
1	12 g	TRANSLUCENT T4
1	50 ml	VITA MODELLING FLUID RS
1	50 ml	VITA OPAQUE FLUID
1	–	BLEACHED SHADE GUIDE 0M1-0M3
1	–	Working instructions

* also available with PASTE OPAQUE (Prod. No. BVMKBCSP)

VITA VMK Master® Sets – VITA classical A1–D4 shades



VITA VMK Master STANDARD SET classical

Standard set for the reproduction of the VITA classical A1–D4 shades
Prod. No. BVMKSETC*

Quantity	Content	Material
1	12 g	WASH OPAQUE W0
16	12 g	OPAQUE A1-D4
16	12 g	OPAQUE DENTINE A1-D4
16	12 g	DENTINE A1-D4
2	12 g	ENAMEL EN1, EN2
1	12 g	TRANSLUCENT T4
1	12 g	WINDOW WIN
3	12 g	CERVICAL CE1-CE3
5	12 g	NECK N1-N5
2	12 g	CORRECTIVE COR1, COR2
1	50 ml	VITA MODELLING FLUID
1	50 ml	VITA MODELLING FLUID RS
1	50 ml	VITA OPAQUE FLUID
1	–	Shade indicator
1	–	VITA classical shade guide
–	–	Accessories
1	–	Working instructions

* also available with PASTE OPAQUE (Prod. No. BVMKSETPC)



VITA VMK Master STARTER SET (A2, A3)

Two-color set
Prod. No. BVMKSSC*

Quantity	Content	Material
2	12 g	OPAQUE A2, A3
2	12 g	OPAQUE DENTINE A2, A3
2	12 g	DENTINE A2, A3
1	12 g	ENAMEL EN1
1	12 g	TRANSLUCENT T4
1	50 ml	VITA MODELLING FLUID RS
1	50 ml	VITA OPAQUE FLUID
2	–	Shade tabs A2, A3
1	–	Working instructions

* also available with PASTE OPAQUE (Prod. No. BVMKSSCP)

VITA VMK Master® Additional sets – suitable for the use with VITA SYSTEM 3D-MASTER® and VITA classical A1–D4 shades



VITA VMK Master ADDITIONAL SET

For natural effects and characteristics
Prod. No. BVMKAS

Quantity	Content	Material
8	12 g	TRANSLUCENT T1-T8
8	12 g	DENTINE MODIFIER DM1-DM8
6	12 g	LUMINARY LM1-LM6
6	12 g	MARGIN MN, M1-M5
1	12 g	OPAL TRANSLUCENT OT1
1	12 g	PEARL TRANSLUCENT PLT1
1	50 ml	VITA MODELLING FLUID
1	–	TRANSLUCENT shade guide
1	–	DENTINE MODIFIER / LUMINARY shade guide
1	–	MARGIN / GINGIVA shade guide



VITA VMK Master GINGIVA SET

Gingival materials with a natural appearance
Prod. No. BVMKGS*

Quantity	Content	Material
5	12 g	GINGIVA G1-G5
2	12 g	GOL, GOD
1	–	MARGIN / GINGIVA shade guide

* also available with PASTE OPAQUE (Prod. No. BVMKGSP)



VITA Modulbox system

All VITA VMK Master sets are shipped in high-quality cardboard boxes.

Upon request, the VITA Modulbox can be ordered separately (for payment).

Complete box No. 2 with two drawers (to be used for VITA VMK Master Standard Set, 10-Color Set): Prod. No. ZMBK2

Complete box No. 1 with one drawer (to be used for VITA VMK Master Additional Set): Prod. No. ZMBK1



Print materials / media

Title	Designation	Order No.
VITA VMK Master	Working Instructions	1645E
VITA VMK Master	Product Brochure	1651E
VITA VMK Master	Product Info	1693E
VITA VMK Master	Combination wheel	1659M
VITA VMK Master	Order form - 3D-MASTER	1654M*
VITA VMK Master	Order form - classical	1655M*

* can be downloaded at www.vita-zahnfabrik.com

VITA VMK Master® Classification tables for VITA SYSTEM 3D-MASTER®
and VITA classical A1–D4 shades

VITA SYSTEM 3D-MASTER shades	OPAQUE	MARGIN	LUMINARY	NECK	CERVICAL	ENAMEL
0M1	OP0	M1	LM1	–	–	EN1
0M2	OP0	M1	LM1	–	–	EN1
0M3	OP0	M1/M2*	LM1/LM2*	–	–	EN1
1M1	OP1	MN/M2*	LM1/LM2*	N1	CE1	EN1
1M2	OP1	M1/M3*	LM2	N1	CE1/CE2*	EN1
2L1.5	OP2	MN/M3*	LM2	N1	CE1/CE2*	EN1
2L2.5	OP2	M2/M3*	LM1/LM3*	N1/N2*	CE2	EN1
2M1	OP2	MN/M2*	LM2	N1	CE1	EN1
2M2	OP2	M2/M3*	LM2/LM3*	N1/N2*	CE2	EN1
2M3	OP2	M3	LM2/LM3*	N1/N2*	CE2	EN1
2R1.5	OP2	M2/M3*	LM2	N1	CE1/CE2*	EN1
2R2.5	OP2	M3	LM2/LM3*	N1/N2*	CE2	EN1
3L1.5	OP3	M2/M5*	LM2/LM6*	N1/N5*	CE2	EN1
3L2.5	OP3	M3/M5*	LM3/LM6*	N1/N3*	CE2/CE3*	EN1
3M1	OP3	M1/M5*	LM2	N1/N5*	CE2	EN1
3M2	OP3	MN/M5*	LM2/LM3*	N2	CE1/CE3*	EN1
3M3	OP3	M3/M5*	LM3	N2/N3*	CE2/CE3*	EN1
3R1.5	OP3	MN/M5*	LM2	N1/N5*	CE2	EN1
3R2.5	OP3	M3/M5*	LM3/LM6*	N2/N3*	CE1/CE3*	EN3
4L1.5	OP4	M5	LM2/LM6*	N1/N5*	CE2	EN1
4L2.5	OP4	M3/M5*	LM3/LM5*	N3	CE1/CE3*	EN1
4M1	OP4	M2/M5*	LM6	N1/N5*	CE2	EN1
4M2	OP4	M5	LM2/LM5*	N1/N5*	CE1/CE3*	EN3
4M3	OP4	M3/M5*	LM3/LM5*	N3/N4*	CE3	EN3
4R1.5	OP4	M2/M5*	LM2/LM5*	N1/N5*	CE2	EN1
4R2.5	OP4	M3/M5*	LM3/LM5*	N3/N4*	CE1/CE3*	EN3
5M1	OP5	M5	LM5/LM6*	N1/N5*	CE1/CE3*	EN1
5M2	OP5	M3/M5*	LM3/LM5*	N3/N5*	CE2/CE3*	EN3
5M3	OP5	M3/M5*	LM3/LM5*	N3/N4*	CE3	EN3

VITA classical A1–D4 shades	OPAQUE	MARGIN	LUMINARY	NECK	CERVICAL	ENAMEL
A1	OP A1	M1/M3*	LM2	N1	CE1	EN1
A2	OP A2	MN/M3*	LM2	N1/N2*	CE1/CE2*	EN1
A3	OP A3	M3	LM3/LM6*	N1/N3*	CE2	EN1
A3,5	OP A3,5	M3/M5*	LM3/LM6*	N3	CE1/CE3*	EN2
A4	OP A4	MN/M5*	LM2/LM5*	N1/N5*	CE3	EN2
B1	OP B1	MN/M2*	LM1/LM2*	N1	CE1	EN2
B2	OP B2	M2/M3*	LM2	N1	CE1/CE2*	EN2
B3	OP B3	M3	LM3/LM6*	N1/N2*	CE2	EN2
B4	OP B4	M3	LM3/LM6*	N1/N3*	CE2/CE3*	EN2
C1	OP C1	MN/M2*	LM2	N1	CE1	EN2
C2	OP C2	M2/M5*	LM2/LM6*	N1/N5*	CE1	EN2
C3	OP C3	M2/M5*	LM6	N1/N5*	CE1/CE2*	EN1
C4	OP C4	M5	LM5/LM6*	N5	CE3	EN1
D2	OP D2	M2/M5*	LM2/LM6*	N1/N5*	CE1/CE2*	EN2
D3	OP D3	M3/M5*	LM6	N1/N5*	CE2	EN2
D4	OP D4	M2/M3*	LM3/LM6*	N1/N5*	CE2/CE3*	EN2






The classifications below may only be used as references!

* Mixing ratio 1:1






VITA VMK Master® Additional materials – to be used for VITA SYSTEM 3D-MASTER® and VITA classical A1–D4 shades

Designation	Description	Prod. No. 12 g
-------------	-------------	----------------

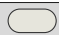
VITA VMK Master NECK

	N1	beige	B4824112
	N2	yellow	B4824212
	N3	yellow-orange	B4824312
	N4	orange	B4824412
	N5	khaki	B4824512


VITA VMK Master TRANSLUCENT

	T1	whitish	B4809112
	T2	yellowish-brownish	B4809212
	T3	pink	B4809312
	T4	neutral	B4809412
	T5	light blue	B4809512
	T6	blue	B4809612
	T7	grey	B4809712
	T8	reddish	B4809812


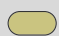

VITA VMK Master OPAL TRANSLUCENT

	OT1	neutral	B4817112
---	-----	---------	----------

VITA VMK Master PEARL TRANSLUCENT







	PLT1	pearly-cream	B4815112
---	------	--------------	----------

VITA VMK Master CERVICAL







	CE1	light yellow	B4820112
	CE2	light orange	B4820212
	CE3	golden yellow	B4820312

Designation	Description	Prod. No. 12 g
-------------	-------------	----------------

VITA VMK Master DENTINE MODIFIER

	DM1	white	B4812112
	DM2	cream	B4812212
	DM3	yellow	B4812312
	DM4	orange	B4812412
	DM5	reddish	B4812512
	DM6	brownish-reddish	B4812612
	DM7	khaki	B4812712
	DM8	grey	B4812812

VITA VMK Master MARGIN

	MN	neutral	B4826012
	M1	white	B4826112
	M2	beige	B4826212
	M3	yellow	B4826312
	M4	orange-reddish	B4826412
	M5	light brown	B4826512

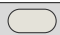

VITA VMK Master LUMINARY

	LM1	white	B4814112
	LM2	sand	B4814212
	LM3	yellow	B4814312
	LM4	light brown-orange	B4814412
	LM5	light brown	B4814512
	LM6	light khaki	B4814612

VITA VMK Master GINGIVA

	G1	dusky pink	B4823112
	G2	orange-pink	B4823212
	G3	pink-red	B4823312
	G4	brown-red	B4823412
	G5	reddish-black	B4823512
	GOL	light	B4827112
	GOD	dark	B4827312

VITA VMK Master CORRECTIVE

	COR1	neutral	B4822112
	COR2	beige	B4822212

VITA