

VITA 3D-Master Interpolated Shades

M Shades

L Shades

Interpolated Shade	3D Mixture	
0M1.5		0M1 : 0M2
0M2.5		0M2 : 0M3
0.5M1		0M1 : 1M1
0.5M1.5	or	0M1 : 1M2
		0M2 : 1M1
0.5M2		0M2 : 1M2
0.5M2.5		0M3 : 1M2
1M1.5		1M1 : 1M2
1.5M1		1M1 : 2M1
1.5M1.5	or	1M2 : 2M1
		1M1 : 2M2
1.5M2		1M2 : 2M2
1.5M2.5		1M2 : 2M3
2M1.5		2M1 : 2M2
2M2.5		2M2 : 2M3
2.5M1		2M1 : 3M1
2.5M1.5	or	2M1 : 3M2
		3M1 : 2M2
2.5M2		2M2 : 3M2
2.5M2.5	or	2M2 : 3M3
		2M3 : 3M2
2.5M3		2M3 : 3M3
3M1.5		3M1 : 3M2
3M2.5		3M2 : 3M3
3.5M1		3M1 : 4M1
3.5M1.5	or	3M1 : 4M2
		3M2 : 4M1
3.5M2		3M2 : 4M2
3.5M2.5	or	3M2 : 4M3
		3M3 : 4M2
3.5M3		3M3 : 4M3
4M1.5		4M1 : 4M2
4M2.5		4M2 : 4M3
4.5M1		4M1 : 5M1
4.5M1.5	or	4M1 : 5M2
		4M2 : 5M1
4.5M2		4M2 : 5M2
4.5M2.5	or	4M2 : 5M3
		4M3 : 5M2
4.5M3		4M3 : 5M3
5M1.5		5M1 : 5M2
5M2.5		5M2 : 5M3

Interpolated Shade	3D Mixture	
2L2		2L1.5 : 2L2.5
2.5L1.5		2L1.5 : 3L1.5
2.5L2	or	2L1.5 : 3L2.5
		2L2.5 : 3L1.5
2.5L2.5		2L2.5 : 3L2.5
3L2		3L1.5 : 3L2.5
3.5L1.5		3L1.5 : 4L1.5
3.5L2	or	3L1.5 : 4L2.5
		3L2.5 : 4L1.5
3.5L2.5		3L2.5 : 4L2.5
4L2		4L1.5 : 4L2.5

R Shades

Interpolated Shade	3D Mixture	
2R2		2R1.5 : 2R2.5
2.5R1.5		2R1.5 : 3R1.5
2.5R2	or	2R1.5 : 3R2.5
		2R2.5 : 3R1.5
2.5R2.5		2R2.5 : 3R2.5
3R2		3R1.5 : 3R2.5
3.5R1.5		3R1.5 : 4R1.5
3.5R2	or	3R1.5 : 4R2.5
		3R2.5 : 4R1.5
3.5R2.5		3R2.5 : 4R2.5
4R2		4R1.5 : 4R2.5